FP0003-2B

Distributed by:
Healthpoint, Ltd.
San Antonio, Texas 78215
1-800-441-8227
www.healthpoint.com

COOK[®] **sis**[™]
Technology
Manufactured by:
Cook Biotech Incorporated

Certified French true equivalent to English version

A
Director of Quality Control
Healthpoint Canada ULC.
2002-06-18

INTENDED USE:

OASIS is indicated for the management of wounds including:

- Partial and full-thickness wounds
- Pressure ulcers
- Venous ulcers
- Chronic vascular ulcers
- Diabetic ulcers
- Trauma wounds (abrasions, lacerations, second-degree burns, skin tears)
- Drainage wounds
- Surgical wounds (donor sites/grafts, post-Mohs' surgery, post-laser surgery, podiatric, wound dehiscence)

OASIS is supplied sterile in peel-open packages and is intended for one-time use.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

CONTRAINDICATIONS: This device is derived from a porcine source and should not be used in patients with known sensitivity to porcine material. This device is not indicated for use in third degree burns.

PRECAUTIONS:

- Do not re-sterilize. Discard all open and unused portions of OASIS.
- Device is sterile if the package is dry, unopened and undamaged. Do not use if the package seal is broken.
- The device must be used prior to the expiration date.
- Discard device if mishandling has caused possible damage or contamination.
- OASIS should not be applied until excessive exudate, bleeding, acute swelling, and infection is controlled.

POTENTIAL COMPLICATIONS: The following complications are possible. If any of these conditions occur, the device should be removed.

- Infection
- Chronic inflammation (Initial application of wound dressings may be associated with transient, mild, localized inflammation.)
- Allergic reaction
- Excessive redness, pain, swelling, or blistering

STORAGE: This device should be stored in a clean, dry location at room temperature.

STERILIZATION: This device has been sterilized with ethylene oxide.

SUGGESTED INSTRUCTIONS FOR USING OASIS

These recommendations are designed to serve only as a general guideline. They are not intended to supersede institutional protocols or professional clinical judgment concerning patient care.

NOTE: Always handle OASIS using aseptic technique.

1. Prepare wound area using standard methods to ensure wound is free of debris and necrotic tissue. An initial surgical debridement of the wound may be necessary to ensure the wound edges contain viable tissue.

2. To apply, cut the dry sheet to a size slightly larger than the outline of the wound area. If the wound is larger than a single sheet, then multiple sheets may be used. Overlap adjoining sheets to provide coverage of the entire wound. For ease of handling, apply OASIS by placing it in a dry state over the wound and rehydrating the sheet using sterile saline or other isotonic solution. Alternatively, rehydrate the sheet by placing it in a bowl of sterile saline or other isotonic solution for at least one (1) minute prior to use.

3. Place the edge of the sheet in contact with the intact tissue. Smooth OASIS into place to ensure the sheet is in contact with the underlying wound bed.

NOTE: If excess exudate collects under the sheet, small openings can be cut in the sheet to allow the exudate to drain.

IMPORTANT: After application, use an appropriate, non-adherent, secondary dressing to maintain a moist wound environment. The optimum secondary dressing is determined by wound location, size, depth, and user preference. Change the secondary dressing as needed to maintain a moist, clean wound area. Frequency of secondary dressing change will be dependent upon volume of exudate produced and type of dressing used. As healing occurs, sections of OASIS may gradually peel and may be removed during dressing changes. Do not forcibly remove sections of OASIS that may adhere to the wound. Alternatively, OASIS may form a caramel-colored gel, which can be rinsed away with gentle irrigation. On inspection, if OASIS is no longer covering the wound, place an additional piece of OASIS over the wound.

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© Cook Biotech Incorporated 2002