


When wounds fail to progress after 2–4 weeks with your standard of care...

Simply introduce OASIS®

OASIS® enables the body to get things moving again.




Application instructions

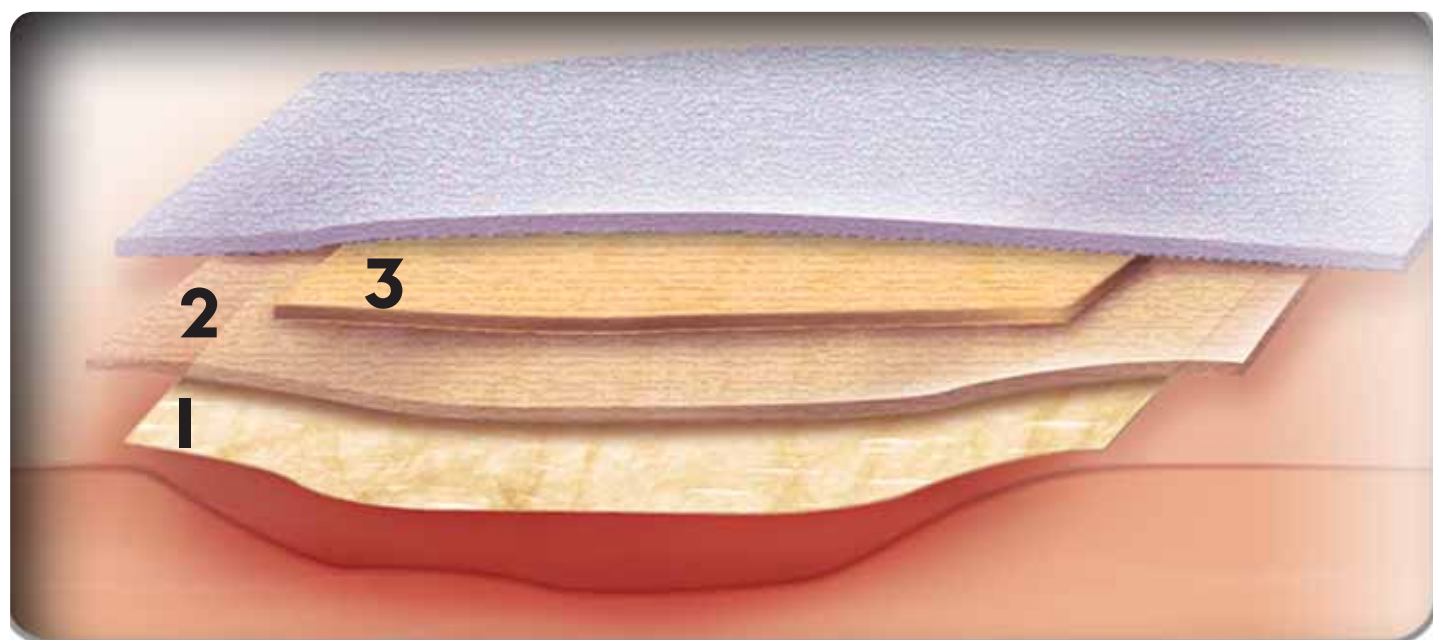

On **wet** wounds

- 1 - OASIS® (hydrate with saline* after placement)
- 2 - Non-Adherent Dressing
- 3 - Absorptive Dressing (eg, calcium alginate)
- 4 - Securement†

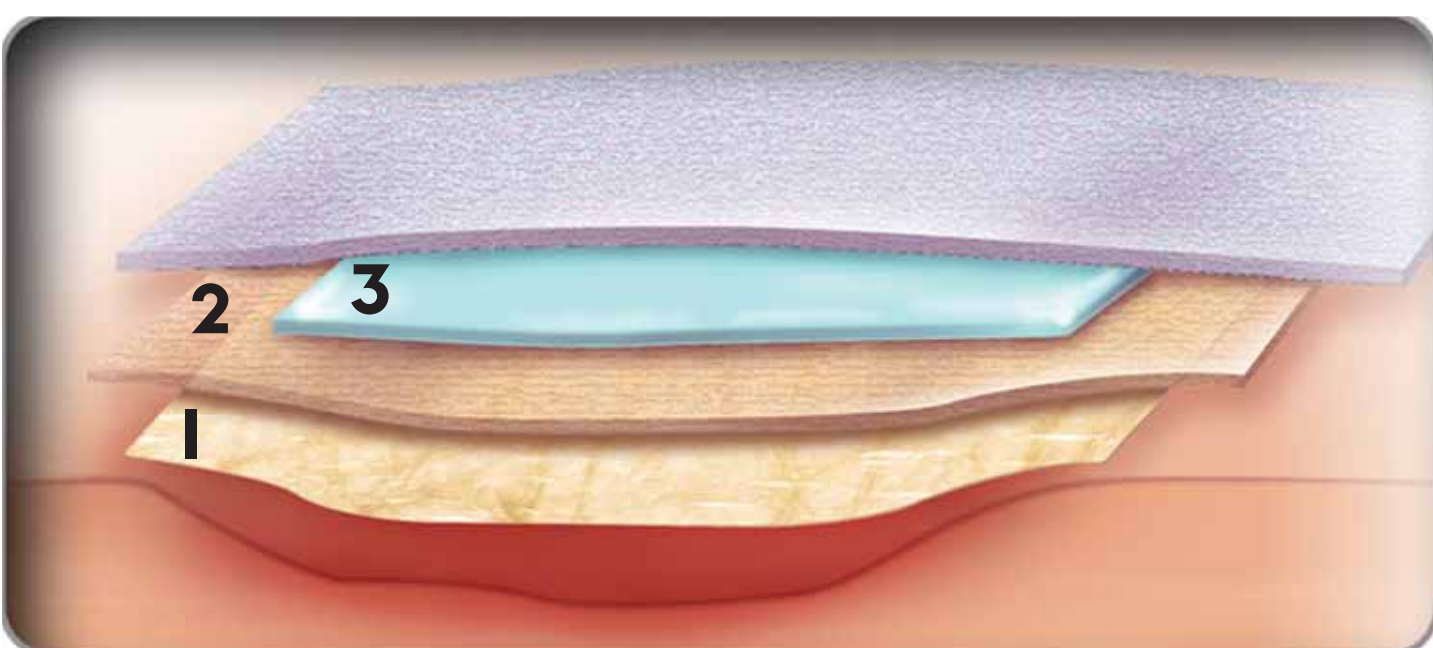

On **dry** wounds

- 1 - OASIS® (hydrate with saline* after placement)
- 2 - Non-Adherent Dressing
- 3 - Hydrogel
- 4 - Securement†

*Or Ringer's Lactate.

†Note: Reimbursement under J734I code requires OASIS® to be anchored with physician's choice of fixation.

Practical application reminders

- **Successful** absorption of OASIS® may form a caramel-colored gel
 - Gently rinse the surface of the wound, leaving the caramel-colored gel intact
 - Apply new OASIS® over absorbed product
- If any of the following conditions occur, use of OASIS® should be discontinued
 - Infection
 - Chronic inflammation
 - Allergic reaction
 - Excessive redness, pain, swelling, or blistering
- **Do not use gauze** to cover OASIS®



INTRODUCE

OASIS®
WOUND MATRIX



Sheet of OASIS® Wound Matrix

Get things moving again

1-800-441-8227
www.healthpoint.com

© Copyright 2006, Healthpoint, Ltd. Printed in USA 137344-0706

OASIS is a registered trademark of Cook Biotech, Inc.

HEALTHPOINT®
A DFB COMPANY